Echo[®] Femoral Hip System



The Echo Femoral Hip System

The Echo Femoral Hip System offers a modern metaphyseal loading, fit and fill design to address increasing hospital demands for both primary and fracture applications. Four stem options with various fixation modes address the distinct needs of each patient, while the customizable instrumentation platform provides enhanced surgical workflow.






Clinically Proven Heritage

The Echo stem options are built upon the heritage of two clinically proven implants: the Integral[®] and Bi-Metric femoral components. Backed by over 20 years of clinical success,¹⁻³ the Echo Bi-Metric stems enhance existing features of each and offer additional benefits to better meet patient needs.

- **100%** Survivorship at 7 years¹ Echo Bi-Metric Femoral Stem 98% Survivorship at 12 years⁴ – Integral Femoral Stem 100% Survivorship at 20 years - Bi-Metric Femoral Stem

Short, Simple and Stable

The Echo Bi-Metric Microplasty stem is a metaphyseal loading, fit and fill femoral prosthesis with a simple, broach only surgical technique. Evolved from the clinically proven Bi-Metric brand³, the Echo Bi-Metric Microplasty option is approximately 30% shorter than the full length stem to conserve bone and accommodate minimally invasive surgical approaches.

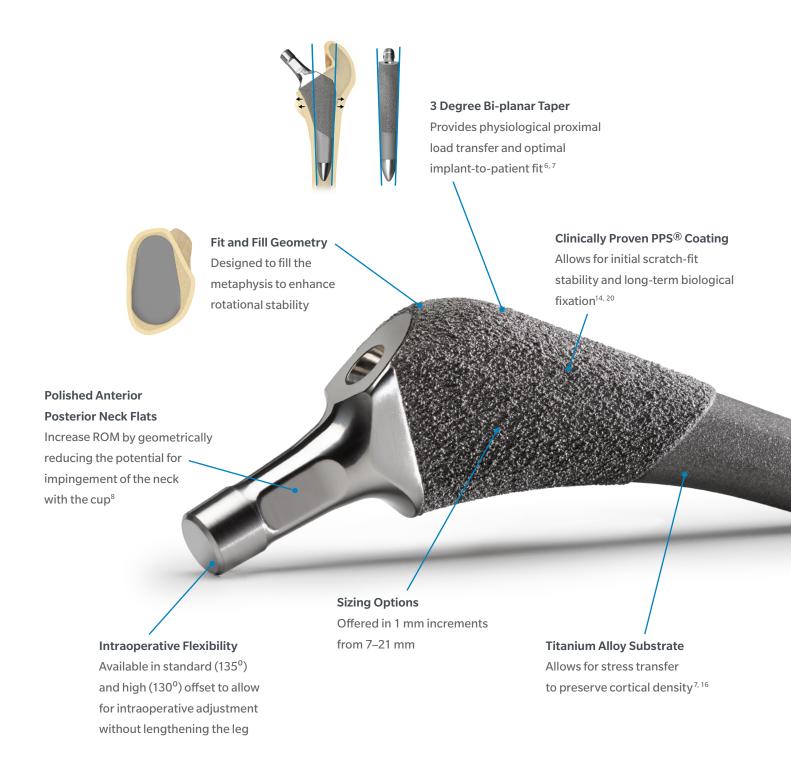

Echo Microplasty Stem Length^{15*}

The Joint Replacement Surgeons of the Indiana Research Foundation in Mooresville, IN conducted strain and micromotion mechanical testing on various stem lengths in an effort to determine the best length for the Echo Bi-Metric Microplasty stem option.

Strain Mechanical Test

Short (96 mm), medium (118 mm) and long (151 mm) Echo Bi-Metric stems were implanted in composite femur models, designed to replicate the mechanical properties of cadaveric specimens, and loaded with the joint reaction force needed to simulate a single-legged stance. Results were then compared to the strain response of an unimplanted femur (control). The test indicated that the short stem prosthesis exhibited loading distribution most closely matching the unimplanted femur.

[µm/m] 4000 3600 3200 2800 2400 2000 1600 1200 800 400 0 Medium Long Short Unimplanted

Digital Image Strain Correlation Analysis (Medial View)

Micromotion Mechanical Test

Short (96 mm), medium (118 mm), long (151 mm) and a reduced lateral shoulder (104 mm) Echo Bi-Metric stem was compared to the commercially available and clinically successful Taperloc[®] Microplasty stem (107.5 mm)⁹⁻¹¹ to determine if stem length and implant design had an effect on primary axial and rotational stability. Digital image correlation was used to track and record the motion as a result of axial and torsional load application in the proximal (P1) and distal (P2) aspect of the porous coating. No statistically significant differences in primary stability were apparent between the various stem lengths.



"This study suggests that, in the patient with high quality proximal bone, short stems may provide more physiological loading without sacrificing primary implant stability."¹⁵

With this in mind, traditional clinical evaluations were conducted to determine the ideal length of the Echo Bi-Metric Microplasty stem. It was determined that the 104 mm stem provided improved guidance into the femoral canal during implantation, while maintaining implant stability.

Short Stem Length

Optimal for minimally invasive surgical approaches and reduces the need for multiple proximal profile options

> Polished Bullet-shaped Distal Tip

Provides a gradual separation from the cortex to reduce distal stresses

Addressing Difficult Anatomies

In addition to its rich clinical heritage, a key benefit of the Echo Bi-Metric full length stem is its two profile options: full proximal profile and reduced proximal profile. The reduced profile option is smaller in the proximal/medial region of the implant by 2 mm compared to the full profile stem. This allows for a full length stem to maximize fit in smaller femoral canals or Dorr Type C femurs, reducing the need to force or undersize the implant to achieve the desired press-fit.

> Fit and Fill Geometry Designed to fill the metaphysis to enhance rotational stability

Clinically Proven PPS Coating

Allows for initial scratch-fit stability and long-term biological fixation^{14, 20}

Polished Anterior Posterior Neck Flats

Increase ROM by geometrically reducing the potential for impingement of the neck with the cup⁸

Intraoperative Flexibility

Available in standard (135°) and high (130°) offset to allow for intraoperative adjustment without lengthening the leg

Proximal Profile Options

Available in full and reduced proximal profiles to accommodate a wide range of femoral canals

3 Degree Bi-planar Taper

Provides physiological proximal load transfer and optimal implant-to-patient fit^{6,7}

> Sizing Options Offered in 1 mm increments from 7–21 mm

Titanium Alloy Substrate

Allows for stress transfer to preserve cortical density^{7, 16}

Polished Bullet-shaped Distal Tip

Provides a gradual separation from the cortex to reduce distal stresses

Complications Addressed with a Reduced Proximal Profile Stem Option



Improperly seated stem

Proper stem seating level

Large Proximal Complications

This illustration depicts an improperly seated stem due to an oversized proximal body in a tight femoral canal



Improperly seated stem Proper stem seating level

Undersizing Complications

This illustration depicts a subsided stem due an undersized implant in a tight femoral canal

Versatile Fracture Treatment

The Echo FX femoral component is a cobalt chrome substrate with an Interlok[®] surface finish, allowing for cemented or press-fit application. Modern design features are included to provide a versatile, dual-purpose implant option for the hip fracture market.

Fit and Fill Geometry

Designed to fill the metaphysis to enhance rotational stability

Reduced Proximal Profile

Optimal fit for Type C or stove pipe canal often associated with elderly hip fracture patients


Polished Anterior Posterior Neck Flats Increase ROM by geometrically reducing the potential for impingement of the neck with the cup⁸

Intraoperative Flexibility

Available in standard (135°) and high (130°) offset to allow for intraoperative adjustment without lengthening the leg

Collared Neck

Transfers stress to the calcar region and reduces subsidence

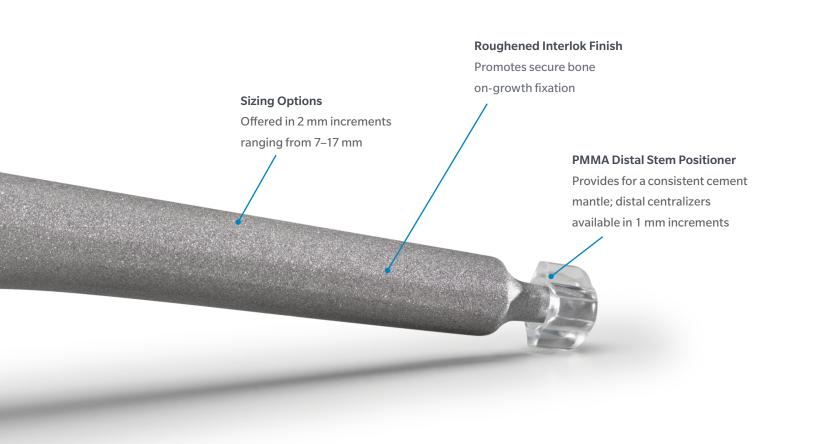

Bone Cement Options

PALACOS[®] Bone Cement

- Long history of clinical success¹⁷
- Excellent handling characteristics
- Formulation with superior antibiotic release^{18**}
- Contrasting green color for high visualization
- ** The FDA has cleared the use of antibiotic loaded bone cement only for use in the second stage of a two stage revision for infection.



HiVac7[™]

- Vacuum Mixing System
- Vacuum mixing in a highly visible cartridge
- Easy to use, few components and simple to assemble
- Safer working environment meets modern safety standards¹⁹



Enhanced Fixation

The Echo PF (Press Fit) femoral component is a titanium alloy, press-fit stem with an Interlok finish and anterior/posterior metaphyseal fins. It is specifically designed for surgeons who prefer to perform hemiarthroplasty or total hip arthroplasty operations while avoiding bone cement. The Interlok surface promotes secure bone on-growth fixation, while the metaphyseal fins allow for additional rotational stability and fixation.

> Fit and Fill Geometry Designed to fill the metaphysis to enhance rotational stability

Reduced Proximal Profile

Optimal fit for Type C or stove pipe canal often associated with elderly hip fracture patients

Polished Anterior Posterior Neck Flats

Increase ROM by geometrically reducing the potential for impingement of the neck with the cup⁸

Neck Angle 4

Available in standard (135°) offset

A/P Metaphyseal Fins Protrude 1 mm to provide

additional rotational stability and fixation

Press-fit Application

Implant is oversized 1.2 mm relative to the broach in the anterior/ posterior region to accommodate a true press-fit

Sizing Options

Offered in 1 mm increments ranging from 7–17 mm

Roughened Interlok Finish

Promotes secure bone on-growth fixation

Titanium Alloy Substrate

Allows for stress transfer to preserve cortical density^{7, 16}

Hemiarthroplasty Options

Zimmer Biomet offers multiple articulation options designed for ease of use during hemiarthroplasty. A wide range of sizes provides the flexibility necessary to address individual patient anatomies.



RingLoc[®] Bi-Polar Acetabular Cup



Endo II Femoral Head

Advancing Total Hip Replacement

From simple primary to complex revision arthroplasty, Zimmer Biomet offers a comprehensive portfolio of hip constructs that combine rich clinical heritage with modern technological advancements. Each of these is designed to address the distinct needs of individual patients, while simplifying surgical workflow.



Echo FX Femoral Stem and RingLoc Bi-Polar Acetabular Cup

> Echo Bi-Metric Microplasty and Continuum[®] Acetabular Total Hip System

* Biomet has evaluated the compatibility of Biomet devices with implants and components from Zimmer orthopedic companies. Only authorized combinations should be used. To determine whether these devices have been authorized for use in a proposed combination with Zimmer products, please contact your sales representative and reference the following website: http://www.product www.productcompatibility.zimmer.com.

References

- 1. Sueyoshi, T. *et al.* Changes in Femoral Stem Geometry Reduce Intraoperative Femoral Fracture Rates in Total Hip Replacement. *Journal of Orthopedics.* (5):115-9, 2015.
- 2. Marshall, A. *et al*. Cementless Titanium Tapered-Wedge Femoral Stem 10- to 15-Year Follow-up. *Journal of Arthroplasty*. 19(5), 2004.
- Meding, J. et al. Twenty-year Follow-up of an Uncemented Stem in Primary THA. Clinical Orthopaedics and Related Research. (473):543-8, 2014.
- Marshall, A.D. *et al.* Cementless titanium tapered-wedge femoral stem—10 to 15-year follow-up. *Journal of Arthroplasty.* 19(5): 546–52, 2004.
- Meding, J. *et al.* Minimum ten-year follow-up of a straight-stemmed, plasma-sprayed, titanium-alloy, uncemented femoral component in primary total hip arthroplasty. *Journal of Bone and Joint Surgery.* 86: 92–7, 2004.
- Rothman, R. et al. Primary Total Hip Arthroplasty with an Uncemented Femoral Component. A Long-Term Study of the Taperloc Stem. Journal of Arthroplasty. 19(2): 151–6, 2004.
- Romagnoli, S. Press-fit Hip Arthroplasty: A European Alternative. Journal of Arthroplasty. 17(4):108–12, 2002.
- Davey J.R. Femoral Offset. http://othonet.on.ca/emerging trends/ notes/Fe=moral%20Offset.htm. (accessed February 15, 2010).
- 9. Barrington, J.W. and Emerson, R.H. The Short and "Shorter of It: Tapered Titanium Stems at 6- to 88-Moth Follow-Up. *The Journal of Arthroplasty*. 28(1):38-40, 2013.
- Christian, C.P. and Jacobs, C.A. Clinical Comparison of THA with a Standard-length or Short Femoral Component. *Orthopedics*. 36(12): e1538-43, 2013.
- 11. Molli, R.G. *et al.* A Short Tapered Stem Reduces Intraoperative Complications in Primary Total Hip Arthroplasty. *Clinical Orthopaedics and Related Research.* 470: 450-61, 2012.

- 12. Widmer, K. and Majewski, M. The impact of the CCD-Angle on Range of Motion and Cup Positioning in Total Hip Arthroplasty. *Clinical Biomechanics*. 20 (7):723–28, 2005.
- Grivas, T.B. et al. Neck Fracture of a Cementless Forged TitaniumAlloy Femoral Stem Following Total Hip Arthroplasty: A Case Report and Review of the Literature. Journal of Medical Case Reports.1:174, 2007.
- McLaughlin, J. et al. Total Hip Arthroplasty with an Uncemented Tapered Femoral Component. Journal of Bone and Joint Surgery.6(90):1290–6, 2008.
- Small, S., Rogge, R. and Berend, M. The Effect of Stem Length on Strain and Micromotion in the Proximal Femur Following Total Hip Arthroplasty. Orthopaedic Biomedical Engineering Laboratory. 63, 2015.
- Head, W. et al. Titanium as the Material of Choice for Cementless Femoral Components in Total Hip Arthroplasty. *Clinical Orthopaedics* and Related Research. (311):85–90, 1995.
- Malchau, et al. Prognosis of total hip replacement: Update and Validation of Results from the Swedish National Hip Arthroplasty Register 1979-1998. Scientific exhibition presented at the 67th Annual Meeting of the American Academy of Orthopaedic Surgeons; March 15-19, 2000; Orlando, USA.
- Meyer, J. et al. Vacuum-Mixing Significantly Changes Antibiotic Elution Characteristics of Commercially Available Antibiotic-Impregnated Bone Cements. Journal of Bone and Joint Surgery. 2011; 93:2049-56.
- 19. Cary, R. *et al*. Methyl Methacrylate: Criteria document an occupational exposure limit. *Health and Safety Executive*.1995.
- Rothman, R. et al. Immediate Weight Bearing after Uncemented Total Hip Arthroplasty. Clinical Orthopedics and Related Research. 349: 156–62, 1998.

*Bench testing is not necessarily indicative of clinical performance.





For complete product information, including indications, contraindications, warnings, precautions and potential adverse events, see package insert and www.zimmerbiomet.com

This material is intended for health care professionals and the Zimmer Biomet sales force only. The distribution to any other recipient is prohibited.

 $\mathsf{PALACOS}^{\textcircled{B}}$ is a trademark of Heraeus Medical GmbH. Under license from Heraeus Medical GmbH, Germany

All content herein is protected by copyright, trademarks, and other intellectual property rights owned by or licensed to Zimmer Biomet or its affiliates unless otherwise indicated, and must not be redistributed, duplicated or disclosed, in whole or in part, without the express written consent of Zimmer Biomet.

© 2015, 2021 Zimmer Biomet



0160.2-US-en-Issue Date-2021-11-10



Legal Manufacturer

Biomet Orthopedics P.O. Box 587 56 E. Bell Drive Warsaw, Indiana 46581-0587 USA

Biomet UK Ltd. Waterton Industrial Estate Bridgend, South Wales CF31 3XA UK

Note: For Ceramic components contained within this brochure Biomet UK, Ltd is the Legal Manufacturer.

www.zimmerbiomet.com