







Spherical Marker Suggestions for ONE Planner™ Hip / ROSA® Hip Imaging Protocol

ONE Planner Hip / ROSA Hip Imaging Protocol is best performed with a **spherical** calibration object on AP view to aid in the templating process. A spherical marker can be purchased or existing spherical marker may be utilized. The spherical marker size must be annotated on the images. The spherical marker is not provided by Zimmer Biomet and must be supplied by the imaging facility.

Equipment, budget and even preferred medical equipment supply companies vary from facility to facility. We are able to provide suggestions, but ultimately the choice is up to your facility.

Considerations:

- Spherical marker must be purchased from a medical supply company
- A size of 25 mm or 25.4 mm (1 inch) spherical marker is most common
- Easy to clean
- Will need a positioning arm or ability to place spherical marker appropriately (at the level of the greater trochanter)

| OrthoMark® by RolleSolutions Inc. | | | | | | | | |
|-----------------------------------|--|--|--|--|---|--|--|--|
| | OM-1001 | OM-1002 | OM-1003 | OM-1005 | OM-1006 | OM-1008 | OM-1009 | OM-1010 |
| |  |  |  |  |  |  |  |  |
| Arm Length | 25" | 25" | 25" | 25" | 10" | N/A | 25" | 25" |
| Arm Weight | 7.5 | 7.5 | 7.5 | 7.5 | 3 | N/A | 7.5 | 7.5 |
| Total Oz | 24 | 13.6 | 16.95 | 13 | 5.75 | 6 | 13 | 14 |
| Min. Radius | 4.5" | 4.5" | 4.5" | 4.5" | 4.5" | N/A | 4.5" | 4.5" |
| Ball Type | Stainless 316 | Stainless 316 | Stainless 316 | Stainless 316 | Stainless 316 | Stainless 316 | Stainless 316 | Stainless 316 |
| Ball Size | 25.4 mm, 1" | 25.4 mm, 1" | 25.4 mm, 1" | 25.4 mm, 1" | 25.4 mm, 1" | 25.4 mm, 1" | 25.4 mm, 1" | 25.4 mm, 1" |
| Base Type | Acrylic | Magnetic | Suction 4" | Clamp | Suction 2" | 8' Strap | Suction 2.75" | Suction 2.9" |
| Hold Time | N/A | Indefinite | 1Hour+ | Indefinite | 1 Minute | Indefinite | 5hrs+ | 5hrs+ |
| Surface Use | Horizontal | Ferrous | Smooth | Up to 2" | Smooth | N/A | Fairly Smooth | Slight Dimple |
| List Price | \$165.00 | \$127.00 | \$155.00 | \$149.00 | \$90.00 | \$72.00 | \$157.00 | \$162.00 |

For additional information regarding discounted price on OrthoMark Spherical Calibration Markers from RolleSolutions, please contact us at personalizedsolutions@zimmerbiomet.com or call/text 574-371-3710.

OrthoMark is a trademark of RolleSolutions Inc.

This material is intended for health care professionals. Distribution to any other recipient is prohibited.

For indications, contraindications, warnings, precautions, potential adverse effects and patient counseling information, see the package insert or contact your local representative; visit www.zimmerbiomet.com for additional product information.

All content herein is protected by copyright, trademarks and other intellectual property rights, as applicable, owned by or licensed to Zimmer Biomet or its affiliates unless otherwise indicated, and must not be redistributed, duplicated or disclosed, in whole or in part, without the express written consent of Zimmer Biomet. ©2021 Zimmer Biomet

