

What Bacteria Does Bactisure® Wound Lavage Work Against?



Technical Brief

Bactisure Wound Lavage is indicated for “cleaning and removal of debris, including microorganisms, from wounds.”¹

By the nature of its design, Bactisure Wound Lavage is effective against a broad spectrum of bacterial species.

Detailed test data can be found in the table below:

	Suspension kill (2 min) (log reduction)	Biofilm removal (2 min, static) (log reduction)	Description	Common associated conditions
<i>Staphylococcus aureus</i>	>5.7	1.1	Gram positive Facultative anaerobe	MRSA, sinusitis, skin infections, pneumonia, meningitis
<i>Staphylococcus epidermis</i>	>5.4	1.9	Gram positive Facultative anaerobe	Skin infections, endocarditis, catheter infections
<i>Pseudomonas aeruginosa</i>	>6.0	3.0	Gram negative Facultative anaerobe	Pneumonia, skin infections, urinary tract infections
<i>C. acnes (aka P. acnes)</i>		2.5	Gram positive Facultative aerobe Difficult to culture	Shoulder surgery infections, acne
<i>C. tropicalis</i>	3.5	n/d	Yeast	Oral thrush, vaginal infections
<i>C. albicans</i>	5.5	n/d	Yeast	Skin, vaginal infections
<i>E. cloacae</i>	>6.2	n/d	Gram negative Facultative anaerobe	Wound infections, urinary tract infections
<i>E. faecalis</i>	>6.0	n/d	Gram positive Facultative anaerobe	Endocarditis, urinary tract infections, meningitis, hospital-acquired infections

¹ FDA 510(k) K161165, Next Science Irrigation Solution

References

- Dr. Garth James, Montana State University Center for Biofilm Education; Next Science report TR-02-14-025.
- Dr. Garth James, Montana State University Center for Biofilm Education; Next Science report TR-02-14-010.
- Antimicrobial Test Labs, Inc. study NG5176.
- Antimicrobial Test Labs, Inc. study NG5062.

Note: Reductions in microbial growth in solution have not been shown to definitively correlate with reductions in infections in patients.

All content herein is protected by copyright, trademarks and other intellectual property rights, as applicable, owned by or licensed to Zimmer Biomet or its affiliates unless otherwise indicated, and must not be redistributed, duplicated or disclosed, in whole or in part, without the express written consent of Zimmer Biomet. This material is intended for health care professionals. Distribution to any other recipient is prohibited. For indications, contraindications, warnings, precautions, potential adverse effects and patient counseling information, see the package insert www.zimmerbiomet.com. Caution: Federal (USA) law restricts this device to sale by or on the order of a surgeon. Rx only.
©2018, 2023 Zimmer Biomet