


iQ™ Intelligent System

The iQ Intelligent Driver is designed to aid in the surgical implantation of Biomet Microfixation 1.5 x 4mm screws.

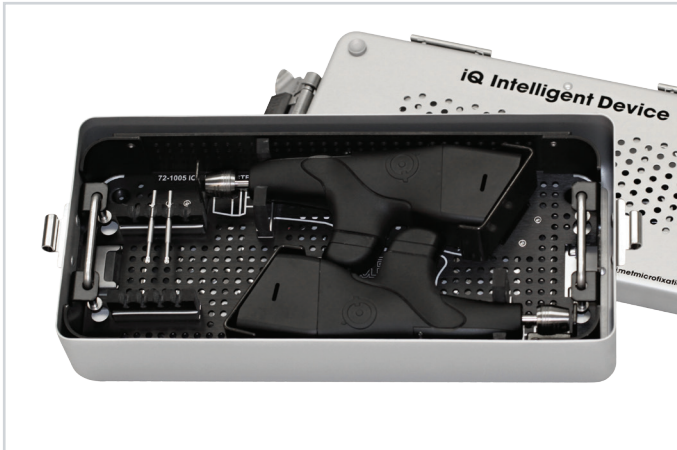

iQ™ Intelligent System Driver Blades and Drills



Cleaning Recommendations

1. Ensure that the driver blade (or drill) and battery have been removed from the device. Discard the battery and drill (if applicable).



WARNING: DO NOT submerge driver. **DO NOT** soak, immerse, or clean in washer/sterilizer.

2. Clean the exterior of the driver and the blades with a damp towelette to remove any debris. All soiled surfaces should be washed with a neutral (7.0) pH detergent. Use a soft nylon brush to remove any visible debris. Carefully rinse the items with or soft tap water.
3. Place driver and blades/drills in the supplied tray and autoclave.

Sterilization Recommendations

1. Sterilization method:
Pre-Vacuum Steam (wrapped):
Temperature: 132°C (270°F)
Dwell Time: 4 minutes
Wrapped Drying Time: 30 minutes minimum
2. Remove from autoclave immediately after the cycle is complete and allow the driver to cool for at least 1 hour.

iQ™ Battery

Battery is packaged sterile. Battery is single use. Dispose after use, DO NOT resterilize.



WARNING: DO NOT sterilize